- Biflu-Jen is a quick drying dental varnish containing a micro-fine suspension of inorganic Fluorides.
  - Its low viscosity allows it to be easily applied to form a thin films and so serve as an insulator of exposed root surfaces.
  - The inorganic Fluorides in Biflu-Jen allow for deep fluoridation when applied to floor of cavities or as a form of prophylaxis against caries and a source of fluoride ions when applied topically to the teeth.
  - Biflu-Jen also contains Thymol as a barrier against local bacterial activity.
  - Biflu-Jen is transparent and therefore very convenient to apply for a dentist and as well for a patient as it does not cause the discoloration of the teeth and dries quickly.
- **INDICATIONS**
- Caries prophylaxis.
  - Fluoridation against caries.
  - Management of exposed root surfaces.
  - Protection of root caries lesion.
  - Local reduction of bacterial activity.
  - Topical application of fluoride following routine scale and polish.
  - Post operative management of acid etched enamel surface.
  - Insulation of tooth surface following mechanical treatment.
  - Cavity liner, protective physical barrier.
  - Dentinal tubule sealant.
- **DIRECTIONS FOR USE**
- Surfaces to be treated are cleaned and thoroughly dried with air.
  - Biflu-Jen must be shaken well before each use. Solid particles must be well dispersed.
  - When applied to the tooth surfaces, simply drip one or two drops of Biflu-Jen onto a foam pellet, and smear a thin layer upon the tooth surface.
  - Avoid forming thick film of varnish as this will tend easily to peel away.
  - For lining of cavities drip one or two drops onto an application brush and then use it to smear a layer of varnish onto walls of the prepared tooth cavity.
- The varnish is allowed to be absorbed for several seconds and then dried with air.
  - As a result Biflu-Jen forms the water impermeable film. With correct application and proper mouth hygiene the film remains in place for several days.
- **COMPOSITION**
- Sodium Fluoride 0.75%
  - Calcium fluoride 0.75%
  - Thymol – 0.5%
  - Natural resins in volatile solvent ad 98%.
- **SUPPLEMENTARY INDICATIONS**
- The application of Biflu-Jen Improved can be repeated two or three times at intervals of 5 – 7 days.
  - Surfaces should be dried and treated with Biflu-Jen again.
- **WARRANTY**
- The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.
- **LIMITATION OF LIABILITY**
- The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.
- **STORAGE**
- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
  - Shelf life – 2 years.
  - DO NOT FREEZE!
- **RECYCLING**
- Dispose of the medical device in accordance with local / regional / national / international legal requirements.
- **PRESENTATION**
- Biflu-Jen – bottle 10 ml.

JenD®

For professional use in dentistry only!

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