

Irrigate and Dry Quickly and Easily

Root Canal Irrigation,
Suction, Drying System

Canal  Clean



Irrigation



Speedy



Air-Driven



Autoclavable


- Saves significant time and effort required for endodontic treatment

Canal Clean allows absolute and effective root canal irrigation, suction and drying in the endodontic treatment just by connecting to a dental unit. The effect of water backflow cleans the root canal effectively, suction and drying will be completed in a few seconds.

Irrigation




Water enters the root canal from the outside of the insert tube. When the insert tube starts suction, water backflow occurs in the root canal and removes the residue in the root canal effectively.

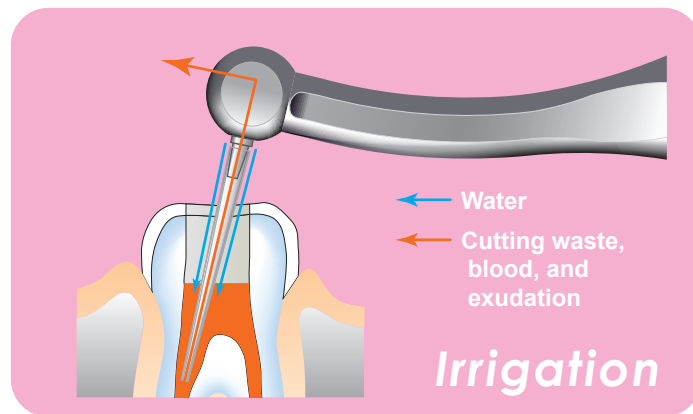

Suction and drying



Stopping water supply allows the insert tube continue suction of residue and remaining water in the root canal. This dries the root canal in just a few seconds.

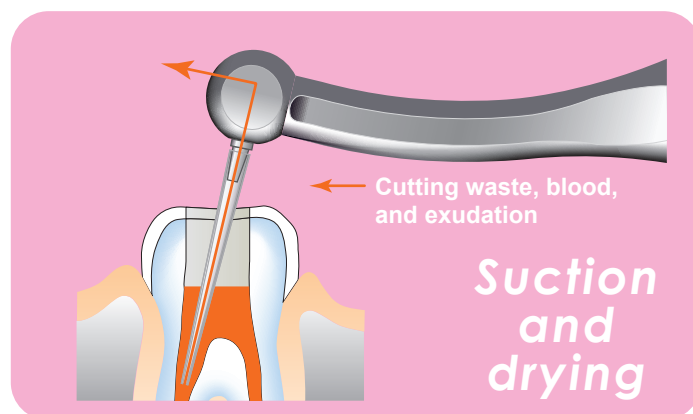

Two choices of tubes - sterilizable and disposable

Insert tube-S

(Slim)



Standard

The insert tubes used in root canals are sterilizable and disposable, it's hygienic. Flexible and slim economical tubes are available.


Insert tube-N

(Flexible)



Option

Easy cleaning and maintenance

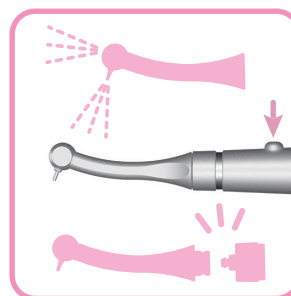

When the insert tube is clogged, a push of the cleaning button removes any residue. The quick-disconnect suction head ensures easy maintenance.

Specifications

Model	Canal Clean-4HA	Canal Clean-2HA	Canal Clean-KA
Backend configuration	Midwest 4-hole	Borden 2 or 3-hole	MULTIflex® coupling
Coupling pressure	0.2 to 0.4 MPa (29 to 58 psi, 2.0 to 4.1 kgf/cm ²)		
Air consumption volume	13 to 23 NL/min at coupling air pressure 0.3MPa		
Water volume	20 mL/min at water pressure of 0.2MPa		
Weight	74 g (2.6 oz)	71 g (2.5 oz)	74 g (2.6 oz)
Length	136 mm (5.4 inch)	130 mm (5.1 inch)	134 mm (5.3 inch)

MULTIflex® is a registered trademark of KaVo Dental GmbH.

Specifications are subject to change without prior notice for improvements.

Manufacturer : **MICRON CORPORATION**

2-17-7, Ikegami, Ota-ku, Tokyo, 146-0082, Japan
Tel : +81 3 3755 0396, Fax : +81 3 5747 5396
http://www.micdent.com/ E-mail : contact@micdent.com

European Authorized Representative : **Advana Ltd.**
Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta