For chemical reaming and cleaning of root canals and exposure of root canals orifices. Chela-Jen Liquid is an effective EDTA solution for chelating, cleaning and debriding of root canals. EDTA reacts with calcium in canal walls helping in reaming process during mechanical instrumentation.

## •• PURPOSE

- Chemical expansion of narrow and hard-to-reach root canals with simultaneous disinfection.
- Detection of canal orifice.
- Unsealing of previously sealed canals.

## •• COMPOSITION

- EDTA 15 g
- Benzalconium chloride 0.3 g
- Excipient Q.S. 100 g

## •• DIRECTIONS FOR USE

Introduce Chela-Jen Liquid into the root canal using a metal cannula or cotton pellet and leave it for 1 – 2 minutes. Low viscosity solution penetrates into the canal readily with fire. Remove softening dentin using appropriate endodontic instrument. Repeat the chemical and mechanical reaming operations till suitable canal preparations is obtained.

In especially difficult cases leave the liquid in the cavity on a cotton pellet for 1 – 2 days capped with a provisional filling.

## •• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

## •• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

## •• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- Refrigeration will further increase shelf life.
- DO NOT FREEZE!

## •• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

## •• PACKING

- Bottle 9 ml.

# JenD<sup>®</sup>

For professional use in dentistry only!

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