Jen-Relief AB is used to lubricate instruments during mechanical or ultrasonic calculus removal procedures. The material has low toxicity, does not cause allergic reactions and chemical burns of the mucous membrane. It is easily brought in hard-to-reach places. Jen-Relief AB is cost effective, easy to use and has a pleasant taste. It has an additional antibacterial effect and reduces tooth sensitivity during mechanical or ultrasonic procedures of calculus removal.

## •• ADVANTAGES

- Application gel based on benzocaine.
- Has antibacterial effect.
- Faster and more profound decrease of sensitivity.
- Does not cause chemical burn of the mucous membrane.
- The drug is applied locally, so that there are no unpleasant sensations that occur after irrigation of the entire oral cavity.
- Ease of application of the material in hard to reach places.
- After using the gel for children fear of using a syringe disappears.

## •• INDICATIONS FOR USE

- Tool lubrication during root canal instrumental procedures.
- Suppression of reflex of mouth closing.
- Reduced sensitivity when removing the excess cement, during installation of a crown or bridge, curettage or the removal of dental plaque.
- Lubrication and decreased sensitivity before applying the rubber dam.

## •• THE METHOD OF APPLICATION

- Use a saliva ejector to remove excess saliva from the patient's mouth.
- When applying the gel with the syringe: remove the protective cap and squeeze the required amount of Jen-Relief AB into place.
- If you use gel from a jar of 30 ml, then apply it with a clean cotton swab.
- Then (after 2 – 3 minutes) infiltration anesthesia is performed.
- The duration of anesthesia is maintained for 10 – 15 minutes.
- If necessary, the drug is applied again.

## •• FLAVORS

- Cherry / Strawberry.

## •• COMPOSITION

It contains 20% benzocaine, which provides fast action, without absorption by soft tissues. And also, it contains flavors.

## •• POSSIBLE COMPLICATION

- Allergic reactions, redness of the mucosa, biting of soft tissues by teeth due to the lack of sensitivity (children), the development of dentophobia (due to an unpleasant feeling of numbness).

## •• PRECAUTIONARY MEASURES

- Do not use Jen-Relief AB for those patients who are allergic or hypersensitive to benzocaine.
- If a rash or other signs of allergic reactions appear on the patient's skin, discontinue use of the material and seek medical attention.
- Do not use for injection.

## •• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

## •• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

## •• STORAGE AND SHELF LIFE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 3 years.
- DO NOT FREEZE!

## •• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

## •• PACKING

- Jen-Relief AB in a bottle of 30 g;
- Jen-Relief AB in 5 ml syringes.

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### EN Jen-Relief AB

Gel lubricant for lubrication of the tool during instrumental or ultrasonic removal of tartar

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