

UltraDry Drying Agent

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : UltraDry is a fast drying, cleaning and degreasing agent for dental surfaces.

Composition : Ethyl alcohol and an organic solvent.

Intended use: It is intended to remove moisture from dental surfaces and to dry tooth surface prior to applying pit and fissure sealants.

Indications for use:

- ◆ To remove moisture from dental surfaces.
- ◆ To dry tooth surface prior to applying pit and fissure sealants (PF Seal, Prevest DenPro Ltd.) and other materials which are adversely affected by the presence of moisture.

Instructions for use :

1. Remove syringe cap and place a red dropper tip on the syringe.
2. Isolate the operative area in the mouth to maintain a clean dry field (for intra oral applications only).
3. Rinse the operative area and air dry with clean, uncontaminated, oil free and moisture free compressed air.
4. Dispense a small amount of UltraDry on the desired surface. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between forefinger and thumb while resting the plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dropper tip for each patient.
5. To promote rapid drying, use a gentle flow of clean, uncontaminated air if desired.
6. The dispensed material shall not be placed back in container and the dose once applied

Warnings : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns or hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components and shall not be reused.

Precautions:

1. UltraDry composition is highly flammable. Handle with care and keep away from open flames.
2. Do not place UltraDry directly on dentin as it can desiccate the dentinal tubules.

Contraindications :

This product is contraindicated for use in or by individuals who have shown hypersensitivity to any of its components.

Storage conditions :

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

2 years from the date of manufacturing.

Presentation :

Ref: 40033

Intro Pack : 1 x 1.2ml Syringes + 5 Applicator Tips.

Ref: 40034

Economy Pack : 1 x 10ml in a Plastic Bottle.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

EC REP
Obelis s.a
Boulevard General Wahis 53
1030 Brussels, Belgium
Tel: +32 2 732 59 54. Fax: +32 2 732 60 63
E-mail : mail@obelis.net

CE
0123

Revision :	06
Date :	08-09-2022

- Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"
- Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"
- SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "FLAMMABLE"
- Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.