Zuremin-CaPF is a two-component oral gel product providing the remineralizing and desensitizing teeth. The product contains a water soluble calcium phosphate salts, potassium nitrate, compounds containing the fluoride ions and some other compounds. When combined upon application to teeth, the Components A and B generates hydroxylapatite and fluoroapatite formation on teeth tissue and provide the desensitizing effect.

#### OUTSTANDING FEATURES OF THE MATERIAL

- Provides the both remineralizing and desensitizing effects.
- Contains anticaries fluoride compounds which correspond to about 1 100 ppm of fluoride ions.
- Does not contain hydrogen peroxide compounds.
- Close to the neutral pH of the material eliminates irritation of the gum tissues.
- The use of the gel composition can lead to a slight lightening of the teeth.
- Sweetened and flavored composition.
- Possesses the antigingivitis activity.
- The remineralization treatment is ideal as a procedure followed by the teeth bleaching.

#### INSTRUCTIONS FOR USE

##### PRE-TREATMENT PROCEDURE

- Before remineralization session, a diagnosis should be performed. The calculus and external stains must be removed. If tissue is traumatized, wait some days before the treatment to minimize the soft tissue trauma caused by tray application.
- Manufacture the soft vinyl tray using the routine laboratory procedure. Instruct patient on the remineralization procedure. Explain treatment regime: teeth cleaning, tray loading, application of Zuremin-CaPF, and tray care. The time required for complete remineralization treatment depends mainly on the level of demineralization or pre-existing carious condition and its severity. It is recommended to evaluate patients every 4-5 days of treatment to determine the duration of remineralization treatment. The appropriate vitamin and micro-mineral diet applied together with remineralization is recommended.

##### PROCEDURE

1. Place approximately equal amount of Zuremin-CaPF Components A and B on a mixing pad.
2. Spatulate the two pastes together for approximately 20 seconds. Make certain that the blend is uniform.
3. The slightly opalescent appearance of mixture indicates the presence of active amorphous calcium fluorophosphate compounds - the evidence that mixture is ready to application.
4. Apply a ready-mixed material to the tray, and place the tray into the mouth. Avoid excessive filling of the tray to prevent the "pumping" ejaculation of material from soft tray while placing into mouth or mouth activity.

5. Remove excess material from mouth.

6. Zuremin-CaPF remineralizes actively 4 – 6 hours and alternative remineralization sessions can be used for as few as half of hour to some hours per day, depending on patient's conditions, level of sensitivity, and day-to-day activities. The most preferable is 1 hour remineralization session per a day.

#### PRECAUTIONS

- Zuremin-CaPF should not be used during pregnancy or when lactating.
- Zuremin-CaPF should not be used if patient has a known allergy or sensitivity to fluorides, nitrates, calcium phosphate salts, glycerin, carbopol, xanthan gum, etc.
- Wait at least one week or more following the remineralization procedure before placing the resin-bonded restoration as the since in this period the adhesion strength of the resin to the dental fabric is significantly reduced.

• **IMPORTANT!** Zuremin-CaPF is used exclusively in mixed form in equal proportions of components A and B. Unmixed components can not be used separately!

#### WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

#### LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

#### STORAGE AND SHELF LIFE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 3 years.
- DO NOT FREEZE!

#### RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

#### PACKING

- Set 2 x 5 ml syringes (Part A and Part B).
- Set 2 x 10 ml syringes (Part A and Part B).

JenD<sup>®</sup>

For professional use in dentistry only!

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EN Zuremin-CaPF

Two-component gel with amorphous calcium phosphate and fluoride for tooth remineralization

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