elite glass

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SOLO PER USO DENTALE

Elite glass is a vinylpolysiloxane especially formulated to make transparent matrices directly in the mouth.

It can be used as a matrix for intra- and extra-oral polymerisation of both photopolymerised and dual polymerisation materials. The use of elite glass highly reduces the oxygen inhibit layer of the composites.

It is thixotropic, very precise, with fast setting times. elite glass is a stable material, with high final hardness: 70 Shore A.

PROPERTIES

Transparent, allows the polymerisation of both photo and dual composites.

Thixotropic, does not drip.

Rigid after setting (70 Shore A).

Accurate.

Dimensionally stable over time and to disinfection.

Can be repositioned in the mouth after setting.

Easy to finish. It can be finished after setting with cutting instruments.

APPLICATIONS

Creation of transparent matrices for the veneering technique.

INSTRUCTIONS FOR USE

Device preparation:

Assemble the device as shown in the drawings (1-2).

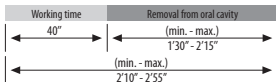
Warning: Before assembling the tip, ensure that the two components (base and catalyst) flow out evenly by applying a light pressure on the dispenser lever and extruding a small amount of material, which must be removed (3).

Subsequently, insert the mixing tip and, if required, the intraoral tip in the cartridge (4).

CREATION OF A TRANSPARENT MATRIX:

- 1) Choose a transparent impression tray (e.g. Zhermack hi tray light clear) of suitable dimensions which can contain at least 2 or 3 mm of elite glass at minimum thickness.
- 2) Apply elite glass onto the impression tray.
- 3) Position the impression tray in mouth until the end of the setting time.
- 4) Remove the impression tray from the mouth, rinse under running water and dry the resulting matrix by gently blowing with air.
- 5) Prepare the tooth or teeth according to the chosen technique.
- 6) Treat the enamel and dentine according to the suggestions of the manufacturer of the chosen composite.
- 7) Apply the composite onto the matrix previously prepared with elite glass.
- 8) Reposition the elite glass matrix in the mouth.
- 9) Polymerise according to the manufacturer's instructions.

TIMES FOR CLINICAL USE

The working time is intended from the beginning of the mixing at 23 °C (73° F), the time in mouth is intended at 35 °C (95 °F)

WARNING: In sensitive subjects, vinylpolysiloxanes may cause irritation or other allergic reactions.

CLEANING AND DISINFECTION OF THE MATRIX

After having been rinsed well under running water, the matrix can be disinfected immediately.

Ideal disinfecting is achieved by immersing the impression in **Zeta 7 solution** (Zhermack) or spraying it with **Zeta 7 spray** (Zhermack). When using other disinfectants, always follow the manufacturer's instructions.

CLEANING AND DISINFECTING THE DISPENSER

The dispenser can be cleaned, disinfected, and sterilised by immersion in liquid or in a steam autoclave. We recommend **Zeta 3** (Zhermack) disinfectant and **Zeta 2** (Zhermack) sterilisation liquid.

STORAGE AND WARRANTY

Store at room temperature.

Elite glass is guaranteed for 36 months if properly stored at temperatures from 5° to 27 °C (41°-80 °F).

WARNINGS

- Contact with latex gloves may prevent vinylpolysiloxane from setting. Avoid direct or indirect contact with latex gloves.
- Vinylpolysiloxane is chemically resistant, so be careful not to stain your clothing.

TECHNICAL DATA (ISO Spec. 4823)

Colour	Transparent
Base/catalyst mixing ratio	1:1
Working time*	40 sec.
Setting time*	5'00"
Detail reproduction	20 µm
Linear dimensional variation after 24h	- 0,18%
Hardness	70 Shore A

*These times are intended from the beginning of mixing at 23 °C (73° F). Higher temperatures shorten, lower temperatures prolong these setting times

IMPORTANT REMARKS: Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the Company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the Company and utilized by the user.

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Zhermack S.p.A., Italy - via Bovazecchino 100 - 45021, Badia Polesine (Rovigo), Italy